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Correo:
Health Plus
Service Center
Dept. 168
3000 Pontiac Trail
Commerce Township,
MI 48390

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Manual Inflate Digital Blood Pressure Monitor

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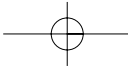
TABLE OF CONTENTS

INTRODUCTORY INFORMATION:

Important Product Notices and Safety Instructions2
About Blood Pressure4
What is Blood Pressure?	
Why Measure Your Blood Pressure?	
Blood Pressure Standard4
How This Blood Pressure Monitor Works6

USAGE INFORMATION:


Name/Function of Each Part7
Display Explanations8
Installing Batteries9
Applying the Cuff10
Measurement Procedure15
Important Notes Regarding Your Blood Pressure Measurement18
Care, Maintenance & Cleaning19
Potential for Radio/Television Interference ..	.21
Troubleshooting22
Specifications23
Five Year Limited Warranty24

IMPORTANT PRODUCT NOTICES AND SAFETY INSTRUCTIONS

When using your blood pressure monitor, basic precautions should always be followed. Please read and follow all instructions and warnings before using this product. Save these instructions for future reference.

- Please note that this is a home healthcare product only and it is not intended to serve as a substitute for the advice of a physician or medical professional.
- Do not use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Consult a healthcare professional for interpretation of pressure measurements. Contact your physician if you have or suspect any medical problem.
- Proper cuff size is critical for accurate measurements. If the index line on the cuff falls outside of the OK range when applied to the user's arm, contact HoMedics Consumer Relations regarding additional cuff sizes.
- This product is not suitable for people with arrhythmias. This device may have difficulty determining the proper blood pressure for individuals with diabetes, poor circulation of blood, kidney problems or for users who have suffered from a stroke.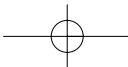

- The pulse display is not suitable for checking the frequency of heart pacemakers.
- Electromagnetic interference: The device contains sensitive electronic components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens). These may lead to temporary impairment of measurement accuracy.
- Use blood pressure monitor only for its intended use.
- Do not wrap the cuff around body parts other than your arm.
- This unit is intended for use in measuring the blood pressure and pulse rate of adults. Do not use this device on infants or those who cannot express their own consent. This unit should not be operated by children.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, *Manual, electronic, or automated sphygmomanometers*.

ABOUT BLOOD PRESSURE

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery walls while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

Why Measure Your Blood Pressure?

Among today's various health problems, those associated with high blood pressure are very common. High blood pressure dangerously correlates with cardiovascular diseases. Therefore, blood pressure monitoring is important for identifying those at risk.

BLOOD PRESSURE STANDARD

The **US Department of Health and Human Services and the National Institutes of Health** have developed a blood pressure standard, that identifies areas of low- and high-risk blood pressure.

This standard, however, is a general guideline as an individual's blood pressure varies among different people, age groups, etc.

	Systolic (mmHg)	Diastolic (mmHg)
Stage 2 Hypertension	≥160	≥100
Stage 1 Hypertension	140~159	90~99
Prehypertension	120~139	80~89
Normal	<120	<80

To determine category of risk when systolic and diastolic readings fall into two areas, use the higher of the two numbers for classification.

It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

For reliable monitoring and reference of blood pressure, keeping long-term records is recommended.


HOW THIS BLOOD PRESSURE MONITOR WORKS

This manual inflate digital blood pressure monitor provides systolic pressure, diastolic pressure and pulse readings quickly and easily. Once the cuff is inflated, the monitor will determine your blood pressures by analysis of your pressure oscillations. When the measurement is completed, the device will deflate the cuff and display your systolic pressure, diastolic pressure and pulse readings.

NAME/FUNCTION OF EACH PART

DISPLAY EXPLANATIONS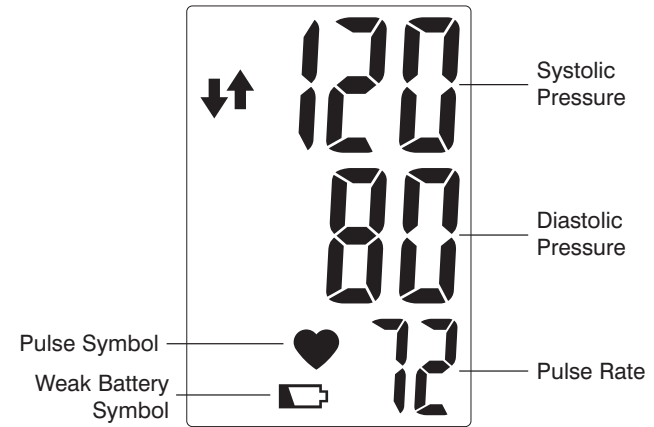


If any of the following letters and numbers appear in the area that diastolic pressure should be displayed, an error has occurred with your reading. See Troubleshooting section of this manual for more information.

EE	Measurement Error: Measure again. Wrap the cuff correctly and keep arm steady during measurement.
E1	Air Circuit Abnormality: Check cuff connection. Measure again.
E2	Pressure Exceeding 300 mmHg: Switch the unit off to clear, then measure again.
E3	Data Error: Remove and reload the batteries.

INSTALLING BATTERIES

1. Press down on latch and lift the battery cover towards you to open the battery compartment.
2. Install or replace 4 “AA” sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery door.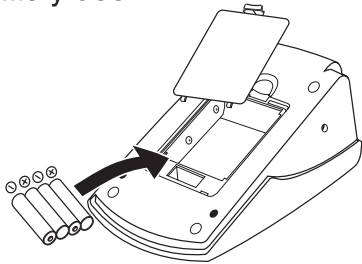

Replace the batteries if:


1. The weak battery symbol appears on the display.
2. Nothing appears on the display when the power is switched on.

Replace the batteries in pairs. Remove batteries when unit is not in use for extended periods of time.


Note: Batteries are hazardous waste. Do not dispose of them together with the household garbage.

APPLYING THE CUFF


Please see the illustration below for a description of the arm cuff.

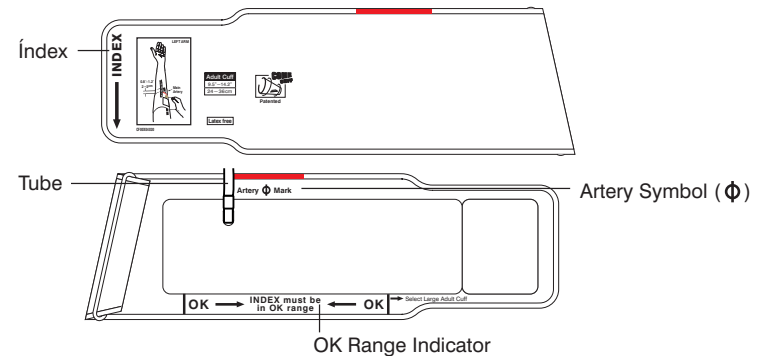1. If the cuff is not assembled, pass the end of the cuff furthest away from the tubing through the metal D-Ring in order to form a loop. The smooth side, without felt material, should be on the inside of the cuff loop.

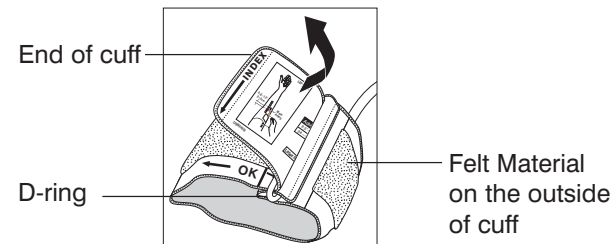
2. Plug the cuff tube into the left side of the unit.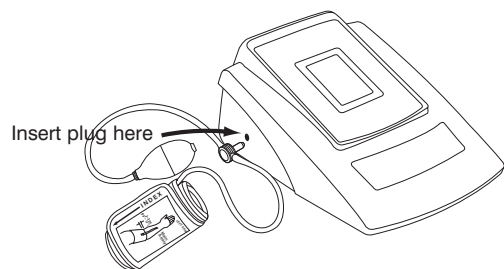


3. Open the arm cuff so that your arm may be placed through it.

4. Remove tight fitting clothing from around your upper left arm. Wrap the cuff around your upper left arm as shown below. The rubber tube should point in the direction of your hand. The bottom edge of the cuff should be 0.8"~1.2" (2~3 cm) above your elbow with the tube positioned over the middle of your arm.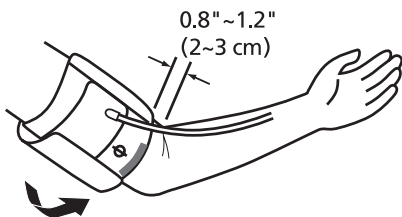


11

5. Locate the main artery by pressing with two fingers approximately 1" (2 cm) above the bend of your elbow on the inside of your left arm. Identify where the pulse can be felt the strongest. This is your main artery.
6. Position the red area of the cuff on the inside of your left arm with the artery symbol (ϕ) over the main artery.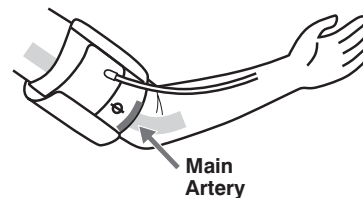


7. To secure the cuff, pull the end toward you, wrap it over the cuff, then press the hook and loop closure together. Allow room for two fingers to fit between the cuff and your arm.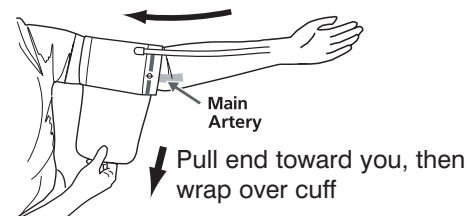

12

Note: If it is not possible to fit the cuff to your left arm, it can also be placed on your right arm. However, all measurements should be made using the same arm. To use on the right arm, you must position the ϕ symbol over the main artery on the inside of your right arm.

Very Important: Proper cuff size is critical to accurate measurement. This cuff is suitable for your use if the Index line falls within the OK range as shown below. If the Index line falls out of the OK range, you will need a cuff with other circumferences. Contact Health Plus Consumer Relations regarding additional cuff sizes.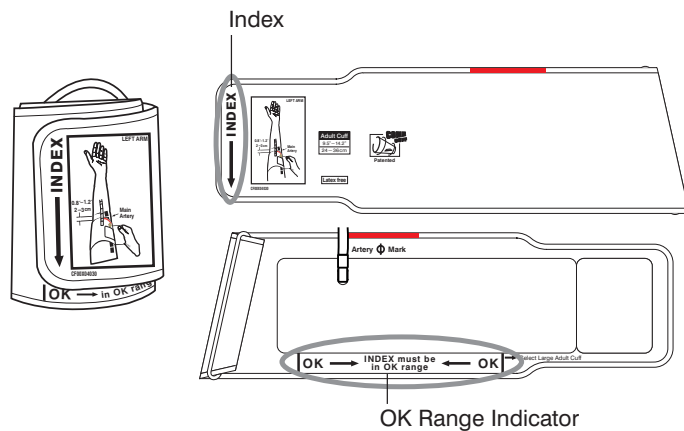

8. Lay your arm on a table (palm upward) so the cuff is at the same height as your heart. Make sure the tube is not kinked.


MEASUREMENT PROCEDURE

Important Notes:

- Blood pressure changes with every heartbeat and is in constant fluctuation throughout the day.
- Blood pressure measurement can be affected by the position of the user, his or her physiologic condition and other factors. For greatest accuracy, wait 1 hour after exercising, bathing, eating, drinking beverages with alcohol or caffeine, or smoking to measure blood pressure.
- Before measurement, it is suggested that you sit quietly for 15 minutes as measurements taken during a relaxed state will have greater accuracy. You should not be physically tired or exhausted while taking a measurement.
- During the measurement, do not talk or move your arm or hand muscles.
- See page 18 for additional notes regarding your blood pressure measurement.

1. With the cuff wrapped around your arm, press the **START/STOP button**. Do not inflate the cuff unless it is wrapped around your arm. All digits will light up to check the display functions. The checking procedure will be completed after about 2 seconds. The ↓ arrow will appear momentarily and then the display will show a “0”. The monitor is **“Ready to Measure”** at this point.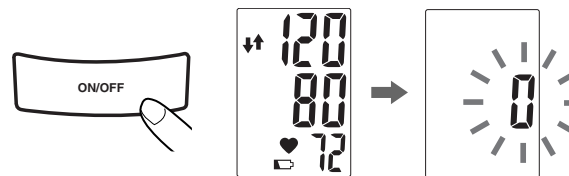

2. Using the hand of the arm that does not have the cuff on it (this will be your right hand if wearing the cuff on your left arm), squeeze the inflation bulb and continue pumping until the pressure reading on the display reaches 170 mmHg. The cuff will then begin slowly deflating as part of the measurement process.

If the pressure in the cuff is not enough, an upward arrow (↑) will appear, directing you to continue pumping. It is recommended that you pump until the pressure reading on the display reaches 200 mmHg.

4. When the measurement is completed, the cuff will deflate entirely. Systolic pressure, diastolic pressure and pulse will be shown simultaneously on the LCD screen.

- Note:** 1. This monitor automatically turns off approximately **1 minute** after last operation. You may also press the **START/STOP button** to turn the unit off.
2. To interrupt the measurement, you may press the **START/STOP button**. The cuff will deflate immediately after a button is pressed.

IMPORTANT NOTES REGARDING YOUR BLOOD PRESSURE MEASUREMENT

- Take your reading in a comfortable environment as measurements can be affected by hot or cold temperatures. Take your blood pressure at normal body temperature.
- Do not move or talk during measurement as this can elevate readings.
- Do not move or cross legs during measurement. Keep feet flat on floor.
- Do not touch cuff or monitor during measurement procedure.
- It is suggested that you take your measurements at the same time each day and use the same arm for consistency.
- Users should wait a minimum of 5 minutes before taking additional measurements. More time may be necessary depending upon your physiology.
- **The measurement results that users receive are for reference only. If users have any blood pressure concerns, please consult a physician.**
- **If inflation reaches 300 mmHg, the unit will deflate automatically for safety reasons.**
- **This product is not suitable for people with arrhythmias.**

- **This device may have difficulty determining the proper blood pressure for users with diabetes, poor circulation of blood, kidney problems, or for users who have suffered a stroke.**

CARE, MAINTENANCE & CLEANING

1. Clean the blood pressure monitor body and cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
2. Leaky batteries can damage the unit. Remove the batteries when the unit will not be used for a long time.
3. If the unit is stored near freezing, allow it to acclimate to room temperature before use.
4. This blood pressure monitor is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems with this device, please contact Health Plus Consumer Relations (contact information can be found on the warranty page).
5. Do not immerse the unit in water as this will result in damage to the unit.

6. Do not subject the monitor and cuff to extreme temperatures, humidity, moisture, and direct sunlight. Protect from dust.
 7. Do not fold the cuff and tubing tightly.
 8. Do not disassemble the monitor or cuff. If in need of repair, refer to the warranty section of this manual.
 9. Do not subject the monitor to extreme shocks (do not drop on floor).
 10. Do not inflate the cuff unless wrapped around arm.
 11. Do not wrap the cuff around body parts other than your arm.
 12. Do not drop or insert any object into any opening or hose.
 13. This monitor may not meet its performance specifications if stored or used outside of these temperature and humidity ranges:
 - **Storage Environment**
 - **Operating Environment**
- | | |
|---|---|
| Temperature:
14°F~140°F (-10°C~60°C) | Temperature:
50°F~104°F (10°C ~40°C) |
| Humidity:
10~90% RH max. | Humidity:
40~85% RH max. |

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE

For USA only:

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

For Canada only:

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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TROUBLESHOOTING

If any abnormality arises during use, please check the following points

SYMPTOMS	POSSIBLE CAUSES	CORRECTION
Unit does not turn on when the START/STOP button is pushed.	Batteries have run down.	Replace them with four new AA alkaline batteries.
	Battery polarities have been positioned incorrectly.	Re-insert the batteries in the correct positions.
EE measurement error symbol shown on display or the blood pressure value is displayed excessively low (or high).	Cuff has been placed incorrectly.	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement? Shaking of the arm with the cuff on.	Measure again. Keep arm steady during measurement.
E1 error symbol shown on display	Air circuit abnormality. Cuff tube may not be plugged into monitor correctly.	Check cuff connection. Measure again.
E2 error symbol shown on display	Inflation pressure exceeding 300 mmHg.	Switch the unit off, then measure again.
E3 error symbol shown on display	Data error.	Remove and reload the batteries.

Note: If the unit still does not work, contact Health Plus Consumer Relations. Under no circumstance should you disassemble or attempt to repair the unit by yourself. Contact information for Health Plus Consumer Relations Department can be found on the warranty page.

SPECIFICATIONS

Power Source:	Four 1.5V DC (AA) batteries
Measurement Method:	Oscillometric
Measurement Range:	Pressure: 40~250 millimeters mercury (mmHg) Pulse: 40~199 beats/minute
Accuracy:	Pressure: ± 3 mmHg Pulse: $\pm 5\%$ of reading
Pressure Sensor:	Semi-conductor
Inflation:	Manual inflation
Deflation:	Automatic Pressure release valve
Auto-shut-off:	1 minute after last button operation
Operation Environment:	Temperature: 50°F~104°F (10°C ~40°C) Humidity: 40~85% RH max.
Storage Environment:	Temperature: 14°F~140°F (-10°C~60°C) Humidity: 10~90% RH max.
Weight:	0.85 lbs (385 g) without batteries
Arm Circumference:	Standard Size Adult Cuff: 9.5"~14.2" (24~36cm)
Dimensions:	4.25"(L) x 6.75"(W) x 2.75"(H) 108mm(L) x 172mm(W) x 70mm(H)
	Type BF Device and cuff are designed to provide special protection against electrical shocks.

Note: These specifications are subject to change without notice.

FIVE YEAR LIMITED WARRANTY (Valid in USA only)

For service or repair, do not return this unit to the retailer. Contact Consumer Relations at:

Mail:
Health Plus
Service Center
Dept. 168
3000 Pontiac Trail
Commerce Township,
MI 48390

Phone:
1-800-466-3342
Business Hours:
8:30am-5pm ET
Monday-Friday

Health Plus sells its products with the intent that they are free of defects in manufacture and workmanship for a period of five years from the date of original purchase, except as noted below.

Health Plus warrants that its products will be free of defects in material and workmanship under normal use and service.

To obtain warranty service on your Health Plus product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address: Health Plus Service Center, Dept. 168, 3000 Pontiac Trail, Commerce Township, MI 48390. No COD's will be accepted.

Health Plus does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate Health Plus in any way beyond the terms set forth herein.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of Health Plus.

This warranty is effective only if the product is purchased and operated in the United States of America. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

5 year
limited warranty

For service or repair, do not return this unit to the retailer. Contact Consumer Relations at:

**Mail:
Health Plus
Service Center
Dept. 168
3000 Pontiac Trail
Commerce Township,
MI 48390**

**Phone:
1-800-466-3342
Business Hours:
8:30am-5pm ET
Monday-Friday**

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OF FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HEALTH PLUS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES.

IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HEALTH PLUS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers.

Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are altered, or modified, without the prior express and written consent of Health Plus.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

©2006 FKA Distributing Co. and its affiliated companies, all rights reserved.
Health Plus™ is a trademark of FKA Distributing Co. and its affiliated companies.
All rights reserved.

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