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(Válida únicamente en los EE.UU.)

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**Instruction Manual and
Warranty Information**

PA-400H

2 year
limited warranty

IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

DANGER- TO REDUCE THE RISK OF ELECTROCUTION:

- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or showering.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink. DO NOT place in or drop into water or other liquid.

WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- DO NOT use outdoors.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.

Caution:
In case of pregnancy, diabetes or illness, consult your doctor before using massager.

Caution:
All servicing of this unit must be performed by authorized HoMedics service personnel only.

- Use heated surfaces carefully. May cause serious burns. DO NOT use over sensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Keep cord away from heated surfaces.
- NEVER block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT use on or near eyes or other highly sensitive areas.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- DO NOT use the percussion action massager directly on one concentrated area of the body. Percussion action massagers are designed to be used in a continuous motion, constantly changing the massage area.
- To disconnect, turn all controls to the "OFF" position, then remove plug from outlet.

SAVE THESE INSTRUCTIONS **Caution — Please read all instructions carefully before operating.**

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time. Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.

**Caution:
Never use the
heat feature
without the
proper
attachment.**

- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children or invalids without adult supervision.
- Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.

Figure 1

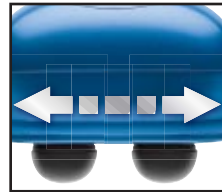
Figure 2

Instructions for Use

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. DO NOT change the plug in any way.

1. This unit is powered by and comes with a UL-approved cord. Untie cord strap, extend cord and insert plug into 120-volt household electrical outlet to power unit.
2. To turn unit on, press POWER button located above handle. A corresponding LED light will illuminate to indicate all selections. (See Figure 1).
3. Choose the desired massage option: steady or program.
4. For steady massage, press MESSAGE button. To increase or decrease the massage intensity level, press the up or down arrows on each side of the MESSAGE button. The intensity level is indicated by the LED screen above the MESSAGE button.
5. For a programmed massage, press the PROGRAM button on the upper left side. Choose any of the three program options:
 - P1 provides slow, steady percussion action to relax and revitalize.
 - P2 provides intense massage action to penetrate and soothe.
 - P3 provides short, quick bursts for energizing stimulation.
 Speed level may be chosen by pressing the SPEED button on the top of the control panel. There are three speed options: L (low), M (moderate) and H (high).
6. Gently apply massage heads to the area you wish to treat, and move slowly in a circular motion.

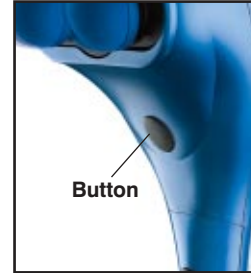
Figure 3

Figure 4

7. Treat the target area for a short period of time, and then move to another area. NEVER treat the same area longer than 3 minutes. CAUTION: As stated in the "IMPORTANT SAFEGUARDS" section of this manual, NEVER use this appliance on or near the eyes, temples or other highly sensitive areas.
8. A massage session of longer than 15 minutes is not recommended.
9. When treatment is completed, turn the unit off and unplug the power supply. Store the unit properly in a cool, dry place.

NOTE: For added protection of your unit, your massager will automatically shut off after 15 minutes

Auto-Spacing Features

The spacing between the massage heads can be adjusted to fit various body frames and areas (See Figure 2). Turn the unit on and simply push the button on the underside of the massager to adjust the spacing between the massage heads (See Figure 3). They will continue to cycle in and out as long as the button is held down. As soon as you find your desired spacing for maximum comfort, release the button.

To Change Attachments

Your massager comes with 6 custom massage head attachments. Each set is a different firmness:

- Hard (light grey)
- Medium (medium grey)
- Soft (black)
- Acu-point (white)
- Heat (red)
- Gel (blue)

To remove the massage heads from the unit, rotate each counterclockwise. Rotate clockwise to fasten the massage heads. (See Figure 4.)

Figure 5

TherapistSelect™ Elite

Programmable Percussion Massager with Adjustable Handle

Figure 6

Handle Adjustability Features

The handle may be adjusted to bend for a more convenient massage of hard to reach body areas (Figure 5). To adjust the handle, pick up the massager with one hand gripping the head and the other hand around the handle. Press the black handle adjustment button (Figure 6) and then turn the handle to the right. Release the button when you have completed your selection. To straighten the handle from the bent position, press the button and turn the handle to the left.

Massage with Thermal Heat

1. Locate the two red (heat) attachments. Attach to the massage head by turning clockwise to secure (See Figure 4).
2. Choose the desired massage option and press the HEAT button.
3. The heat feature will work only in conjunction with the heat massage attachments. The heat indicator will illuminate when used with other attachments, but will not be felt.

NOTE: To eliminate the risk of being burned, never use the heat function without the heat attachments. DO NOT use the thermal heat continuously for more than 15 minutes. DO NOT use the thermal heat if you have sensitive skin, poor circulation, discolored areas, swelling, burning, inflammation, or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.

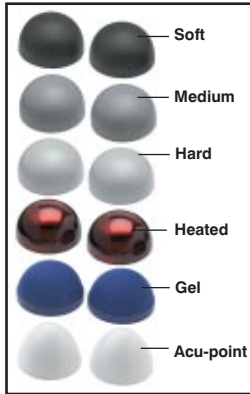
Figure 7

Caution:
Never use the heat feature without the proper attachment.

Maintenance

To Clean

- Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. NEVER allow water or any liquids to come into contact with the handle.

To Store

- Unplug the appliance from the outlet and allow it to cool before storing in its box or a clean dry place. Coil the cord and secure it with the supplied fastener. DO NOT hang the unit by the powercord.

Manual or program massage options

3 Different Speed Levels
With massage programs

3 Program Massage Functions

7 Massage Intensities
With steady massage

Optional Heat
To soothe tired muscles

Dual Pivoting Heads
For invigorating massage

Adjustable Spacing
To fit all body areas

6 Custom Massage Attachment Sets Included:
Heat, soft, medium, hard, Acu-point, and gel

Bonus Storage Bag Included:
For massager & attachment sets

TWO YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

Mail To:

HoMedics
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

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