ATLAS™ VIBRATION ACU-NODE MASSAGER

INSTRUCTION BOOKLET AND SAFETY INFORMATION

SR-BALL

FIND MORE HELP ONLINE

Visit the URL to the right to find how-to photos, usage tips, and additional information about this product.

www.homedics.com/atlas

INSTRUCTIONS FOR USE

- **Prior to use:** Twist the massager halves in opposite directions to access the internal battery compartment (Fig. 2). Remove and dispose of the small plastic tab.
- Press the power button to activate the massager (Fig. 1). With plastic tab removed, massager will continue to run without holding down the power button.
- Hold the massager over the desired area to be massaged.
- After completing your massage, press the power button to turn the massager off.
- When the batteries need replacing, twist the massager halves in opposite directions to access the battery compartment (Fig. 2). Use a coin to open battery chamber and insert 2 “AAA” batteries as directed in the compartment. Close the battery door and reattach the top and bottom halves of the massager. Twist to close. Turn the halves only until snug — do not over-turn or use excess force.

FIG 1

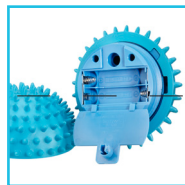
FIG 2

TO CLEAN

Clean only with a soft fabric cloth. Never allow water or other liquids to come into contact with this unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean.

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING THIS APPLIANCE, BASIC PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

- This product is not intended for medical use.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics, specifically any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- Not for use by children.
- Turn off appliance when not in use.
- Never operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics service center for examination and repair.
- Keep away from heated surfaces.

SAVE THESE INSTRUCTIONS

CAUTION: PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- If you have concerns regarding your health, consult your doctor before using this product.
- Those with the following conditions should consult their doctor before using this product:
 - Pregnancy
 - Malignant tumor
 - Diabetes
 - Conditions requiring rest
 - Heart disease
 - Pacemakers
- If at any time soreness, rash, or pain occurs, discontinue use and consult your physician.
- NEVER use this product directly on swollen or inflamed areas, skin eruptions, or an open wound.
- A massage session longer than 20 minutes is not recommended.
- **Attention California Residents:**
WARNING: This product contains chemicals known to the State of California to cause cancer, birth defects, and other reproductive harm.

BATTERY PRECAUTIONS

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of batteries may cause damage to the unit.
- Do not mix old and new batteries.
- Do not mix alkaline, standard (carbon-zinc) or rechargeable (ni-cad, ni-mh, etc.) batteries.
- Do not dispose of batteries in fire. Batteries may explode or leak.
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove depleted batteries promptly.
- Dispose of properly in accordance with all state, province, and country regulations.

This product contains small parts that may present a choking hazard for young children.

Caution: All servicing of this massager must be performed by authorized HoMedics Service Personnel only.

90 DAY LIMITED WARRANTY

For detailed warranty instructions and information, visit: http://media.homedics.com/warranty/warranty_90days.pdf

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