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(Válida únicamente en los EE.UU.)

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IB-BKROLA

**Instruction Manual and
Warranty Information**

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**1 year
limited warranty**

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IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

CAUTION- PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE APPLYING.

- If at any time soreness, rash, or an increase in pain occurs, discontinue use and consult your physician.
- Only use this product for its intended use as described in the manual.
- Do not use this product over an open wound.
- This is not a toy. Children should not use or play with this product.




Caution:
**All servicing of
this massager
must be
performed
by authorized
HoMedics
service
personnel only.**

SAVE THESE INSTRUCTIONS

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should never use this product.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.

Battery Precautions

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of batteries may cause damage to the unit.
- Do not mix different types of batteries together (e.g., alkaline with carbon-zinc or old batteries with new ones).
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.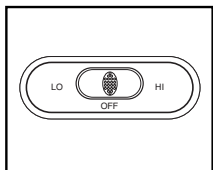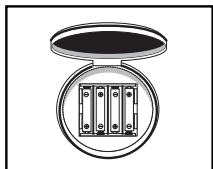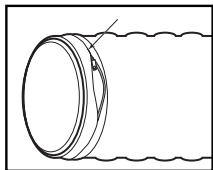

Instructions For Use

1. Before use, install batteries. This unit requires 4 "AA" cell batteries-not included. Zipper is located on the end of the unit. Unzip to expose battery compartment. Insert batteries as illustrated and close zipper.
2. Locate ON/OFF switch along side of the unit. Slide the switch toward "HI" or "LO" to turn unit on. Test roll after installing batteries. If unit fails to turn on, check battery polarity for correct positioning.
3. Position roll on area of the body needing massage therapy.

Maintenance


To Store

Place roll in its box or in a safe, dry cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface.

To Clean

Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

MassageComfort

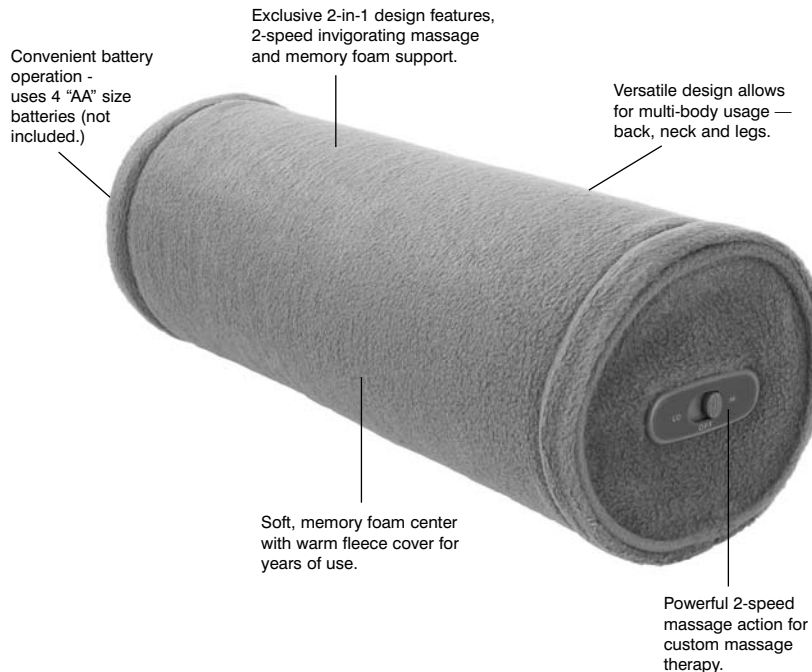1 YEAR LIMITED WARRANTY
(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of 1 year from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of one year from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

©2005 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies. The beauty of living well™ and HoMedics Mobile Comfort are trademarks of HoMedics Inc. and its affiliated companies.

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All rights reserved.